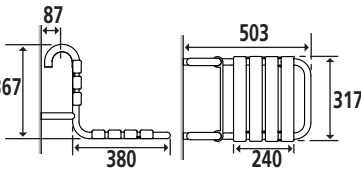
 Poids maximum utilisateur / Max. user weight / Maximumgewicht gebruiker / Peso máximo utilizador / Peso max. Utente/Peso máximo do utilizador / Maximals Benutzergewicht / Макс. вес пользователя.
 Poids du produit / Product's weight / Productgewicht / Peso del producto / Peso del prodotto / Peso do produto / Gewicht des Produktes / Вес изделия

Référence P / Reference P / Referentie P / Referencia P / Codice P / Referência P / Referenz P / Код P :		047534	
Fonction	Permet de prendre une douche en position assise.		
Function	Allows to take a shower in a seated position.		
Functie	Zo kan u zittend een douche nemen.		
Función	Permite tomar una ducha en posición sentada.		
Funzione	Permette di prendere una doccia in posizione seduta.		
Função	Permite tomar um duche sentado.		
Funktion	Ermöglicht es, im Sitzen zu duschen.		
Назначение	Предназначено для принятия душа в положении сидя.		
Descriptif			90 kg
Description			3,90 kg
Beschrijving			
Descripción			
Descrizione			
Descrição			
Beschreibung			
Описание			
Numéro notice / Instruction sheet No. / Fichenummer / Número del manual / Numero manuale / Número do manual / Gebrauchsanweisungsnummer / Номер инструкции		S00001336	
Date de création / Creation date / Aanmaakdatum / Fecha de creación / Data di creazione / Data de criação / Erstellungsdatum / Дата создания		11/12/2002	
Révision n° / Revision No. / Revisie nr. / Revisión n° / Revisione n° / Revisão n° / Revisionsnr. / № редакци		07	
Date / Date / Datum / Fecha / Data / Datum / Дата		02/06/2015	

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GB  **Do not install this equipment before reading the following instructions:**  **Fitting instructions.** Grab bar not supplied. The shower seat has to be fitted on a grab bar; length from 600 to 800 mm. Before drilling into the wall, physically

hold the product against the wall so that the grab bar is PARALLEL to the floor, and at 80 cm from the floor. Then mark the position of the holes to be drilled. Plugs have to be selected according to the type of wall. The 6 fixings points have to be necessarily respected.  **Care and maintenance. IMPORTANT:** In order to guarantee the lasting quality of this product and its safety for the user, we recommend that when cleaning it you should use mild soap solutions. **CAUTION: DO NOT USE** cleaning products made of: etones, hydrocarbons, acids, bases, esters and ethers. The disinfection has to be made according to the recommendations of use of the various products of disinfection. After use of a cleaning product, make a rinsing in the clear water.  **Safety warnings.** The shower seat adds security to the user if it is fitted according to instructions. After fitting and before using, make sure that the fixing screws are tightened securely. The shower seat is designed to be used in seated position. To guarantee the user's safety, it is not advised to rise up on the seat. To avoid any pinching risks, do not put fingers between grab bar and shower seat. The product does not contain any hazardous substances. The manufacturer shall have no liability for any injury caused by improper installation or use of this shower seat.  At the end of its life cycle, the user must have the product recycled by a certified facility in the country of use.

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Fabricant : GODONNIER SAS