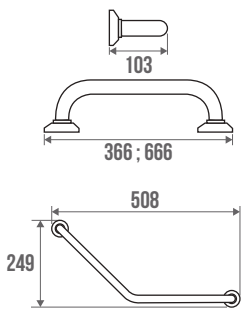
Référence P / Reference P / Referentie P / Referencia P / Codice P / Referência P / Referenz P / Код P :		049135	049165	049225	
		049136	049166	049226	
		049137	049167	049227	
Fonction	Permet à l'utilisateur de prendre appui en toute sécurité.				
Function	Allows the user to lean safely.				
Functie	Zo kan de gebruiker in alle veiligheid ondersteund worden.				
Función	Permite al usuario apoyarse con seguridad.				
Funzione	Permette di prendere appoggio con sicurezza.				
Função	Permite ao utilizador apoiar-se em toda a segurança.				
Funktion	Zum sicheren Abstützen des Benutzers.				
Назначение	Служит для пользователя надежной опорой.				
Descriptif Description Beschrijving Descripción Descrição Beschreibung Описание				150 kg	
			049135 049136 049137	0,380 kg	
			049165 049166 049167	0,520 kg	
			049225 049226 049227	0,480 kg	
		Numéro notice / Instruction sheet No. / Fichenummer / Número del manual / Numero manuale / Número do manual / Gebrauchsanweisungsnummer / Номер инструкции			A49130012
		Date de création / Creation date / Aanmaakdatum / Fecha de creación / Data di creazione / Data de criação / Erstellungsdatum / Дата создания			14/03/2011
		Révision n° / Revision No. / Revisie nr. / Revisión n° / Revisione n° / Revisão n° / Revisionsnr. / № редакции			04
		Date / Date / Datum / Fecha / Data / Data / Datum / Дата			05/12/2017

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GB  **Do not install this equipment before reading the following instructions:**

 **Fitting instructions.** Advised height 80cm from the ground. Straight grab bars can be installed obliquely (135°) or horizontally. Before drilling into the wall, physically hold the product against the wall and mark the position of the holes to be drilled. Plugs have to be selected according to the type of wall. The 6 fixings points have to be necessarily respected.  **Care and maintenance. IMPORTANT:** In order to guarantee the lasting quality of this product and its safety for the user, we recommend that when cleaning it you should use mild soap solutions. **CAUTION: DO NOT USE** cleaning products made of: etones, hydrocarbons, acids, bases, esters and ethers. After use of a cleaning product, make a rinsing in the clear water.  **Safety warnings.** Grab bars add security to the user if they are fitted according to instructions. After fitting and before using, make sure that the fixing screws are tightened securely. Grab bars are not designed to support the total weight of the user, use it for assistance ONLY. DO NOT use grab bar as a step to reach area, do not sit or hang on grab bar. **Use it for assistance ONLY.** The product does not contain any hazardous substances. The manufacturer shall have no liability for any injury caused by improper installation or use of this grab bar.  At the end of its life cycle, the user must have the product recycled by a certified facility in the country of use.

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Fabricant : GODONNIER SAS