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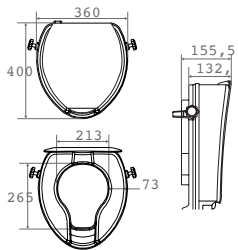
GB  **Do not install this equipment before reading the following instructions:**

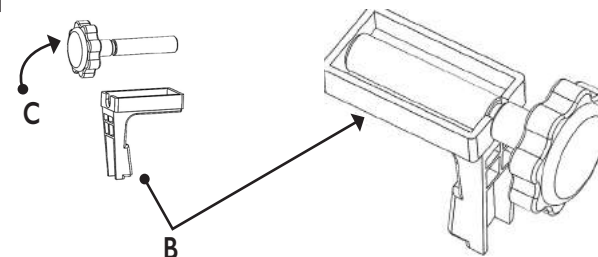
 **Fitting instructions.** Insert the fixing wheel into the support bracket. Snap the support bracket into the dedicated parts below the raised toilet. Put physically the product on the WC bowl and screw to adjust correctly.

 **Care and maintenance. IMPORTANT:** In order to guarantee the lasting quality of this product and its safety for the user, we recommend that when cleaning it you should use mild soap solutions. **CAUTION: DO NOT USE** cleaning products made of: etones, hydrocarbons, acids, bases, esters and ethers. After use of a cleaning product, make a rinsing in the clear water.

 **Safety warnings.** The raised toilet seat adds security to the user if it is fitted according to instructions. After fitting and before using, make sure that the fixing wheels are tightened securely. The toilet seat is designed to be used in seated position. To guarantee the user's safety, it is not advised to rise up on the seat. To avoid any pinching risks, do not put fingers in the mobile parts. The product does not contain any hazardous substances. The reseller shall have no liability for any injury caused by improper installation or use of this toilet seat.

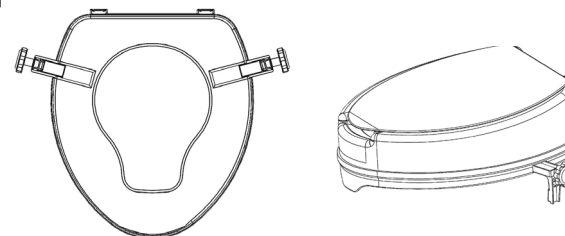
At the end of its life cycle, the user must have the product recycled by a certified facility in the country of use.

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