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

 **Do not install this equipment before reading the following instructions:**  **Fitting instructions.** Recommended height for the horizontal part : 80 cm from the ground. Before drilling into the wall, physically hold the product against the wall and mark the position of the holes to be drilled. Plugs have to be selected according to the type of wall. The fixings points have to be necessarily respected.  **Care and maintenance. IMPORTANT:** In order to guarantee the lasting quality of this product and its safety for the user, we recommend that when cleaning it you should use mild soap solutions. **CAUTION: DO NOT USE** cleaning products made of: etones, hydrocarbons, acids, bases, esters and ethers. After use of a cleaning product, make a rinsing in the clear water.  **Safety warnings.** Grab bars add security to the user if they are fitted according to instructions. After fitting and before using, make sure that the fixing screws are tightened securely. **DO NOT** use grab bar as a step to reach area, do not sit or hang on grab bar, use it for assistance **ONLY**. **The product does not contain any hazardous substances. The manufacturer shall have no liability for any injury caused by improper installation or use of this grab bar.**  At the end of its life cycle, the user must have the product recycled by a certified facility in the country of use.


Vous aurez besoin / You will need

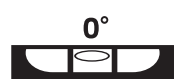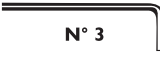						
0°			TORX T25	N° 3		

A


ENTRETOISES DEMONTAGE / DISMOUNTING SPACER

1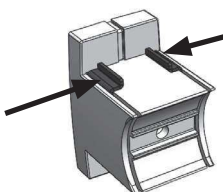

Démonter l'entretoise de son support
Remove the dismounting spacer from its bracket

2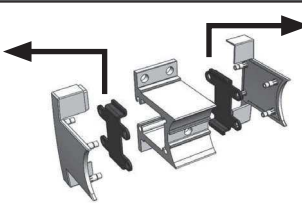


Jeter l'entretoise, se reporter au consigne de recyclage local
Discard spacer, refer to local recycling locker

B

FIXATION / ATTACHMENT

1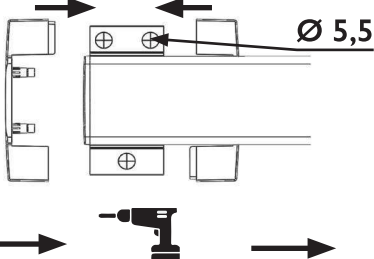


Ø 5,5

1- Marquer l'entraxe
Mark the center distance

2- Percer le mur
Drill the wall

3- Visser la platine
Screw the wall plate

2

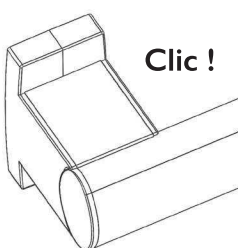Clic !

Fabricant : PELLET ASC

ZI de Chapotin - 240 rue des Frères Voisin - CS 270048 - 69967 Chaponnay Cedex - France
Tél. +33 (0)4.78.96.82.20 - Fax +33 (0)4.78.96.70.52 - www.pellet-asc.fr - info@pellet-asc.fr