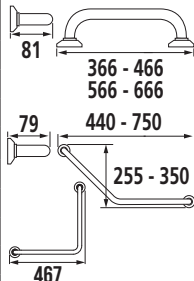
Référéncé G / Reference G / Referentie G / Referencia G / Codice G / Referência G / Referenz G / Код G :		549801 - 549831		
Référéncé P / Reference P / Referentie P / Referencia P / Codice P / Referência P / Referenz P / Код P :		010203 - 010204 - 010205 010206 - 043293 - 043532 043533		
Fonction Function Functie Función Funzione Função Funktion Назначение	Permet à l'utilisateur de prendre appui en toute sécurité. Allows the user to lean safely. Zo kan de gebruiker in alle veiligheid ondersteund worden. Permite al usuario apoyarse con seguridad. Permette di prendere appoggio con sicurezza. Permite ao utilizador apoiar-se em toda a segurança. Zum sicheren Abstützen des Benutzers. Служит для пользователя надежной опорой.			
Descriptif Description Beschrijving Descripción Descrizione Descrição Beschreibung Описание			010203 - 010204 043532	150 kg
			549801 - 549831	
			043293 - 043533	120 kg
			010205	80 kg
			010206	70 kg
			010203 - 549831	0,40 kg
			010204	0,47 kg
			010205	0,54 kg
			010206	0,61 kg
			043293	0,71 kg
043532 - 549801		0,49 kg		
043533		0,73 kg		
Numéro notice / Instruction sheet No. / Fichenummer / Número del manual / Numero manuale / Número do manual / Gebrauchsanweisungsnummer / Номер инструкции			A01020305	
Date de création / Creation date / Aanmaakdatum / Fecha de creación / Data di creazione / Data de criação / Erstellungsdatum / Дата создания			14/03/2011	
Révision n° / Revision No. / Revisie nr. / Revisión n° / Revisione n° / Revisão n° / Revisionsnr. / № редакции			09	
Date / Date / Datum / Fecha / Data / Data / Datum / Дата			09/06/2021	

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Do not install this equipment before reading the following instructions:

Fitting instructions. Advised height 80 cm from the ground. Straight grab bars can be installed obliquely (135°) or horizontally. Before drilling into the wall, physically hold the product against the wall and mark the position of the holes to be drilled. Plugs have to be selected according to the type of wall. The fixings points have to be necessarily respected.

Care and maintenance. IMPORTANT: In order to guarantee the lasting quality of this product and its safety for the user, we recommend that when cleaning it you should use mild soap solutions. **CAUTION: DO NOT USE** cleaning products made of: etones, hydrocarbons, acids, bases, esters and ethers. After use of a cleaning product, make a rinsing in the clear water. **⚠ Safety warnings.** Grab bars add security to the user if they are fitted according to instructions. After fitting and before using, make sure that the fixing screws are tightened securely. **DO NOT** use grab bar as a step to reach area, do not sit or hang on grab bar, use it for assistance **ONLY**. The product does not contain any hazardous substances. The manufacturer shall have no liability for any injury caused by improper installation or use of this grab bar. **⚠** At the end of its life cycle, the user must have the product recycled by a certified facility in the country of use.

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